# Energel™

Wound Hydrogel

Compatible  
with **Advanced  
Microcurrent  
Technology™**

- EN** Instructions for Use
- ES** Instrucciones de uso
- FR** Mode d'emploi
- PT** Instruções de uso

Manufactured for: / Fabricado para: /  
Fabriqué pour: / Fabricado para:  
**Vomaris Wound Care, Inc.**  
1911 East Fifth Street  
Tempe, AZ 85281 USA  
(866) 496-8743 (480) 921-4948  
www.vomaris.com

© 2017 Vomaris. All rights reserved. Energel and Advanced  
Microcurrent Technology are trademarks of Vomaris Innovations, Inc.

IFU-33 Rev. A

## Energel™

Wound Hydrogel

*Compatible with Advanced Microcurrent Technology™*

### Description

Energel™ wound hydrogel is a sterile and water-soluble gel formulated to maintain a moist wound environment and provide moisture to a dry wound. Energel wound hydrogel may be used to moisten Advanced Microcurrent Technology dressings.

### Indications

Energel wound hydrogel is intended to cover a wound, to protect against abrasion, friction, desiccation, and contamination.

### Application

1. Cleanse wound and surrounding skin prior to application.
2. Apply a thin even layer of Energel wound hydrogel to wound dressing surface and place the dressing onto the wound bed.
3. Apply appropriate cover dressing(s) to help manage wound drainage and maintain a moist wound environment.

### Removal and Disposal

- Gently remove gel from wound site.
- Discard after single use.

### Contraindications

None known.

### Storage

- Store in a cool dry place.
- Store away from direct sunlight.

### Warnings and Precautions

- For external use only.
- Bleeding should be controlled before application of cover dressings.
- Do not use on patients with known sensitivity to parabens.
- Discard product if packaging is damaged or evidence of contamination is present.
- Single use only. Once opened, this product should not be reused due to risk of contamination.

## Energel™

Hidrogel para heridas

*Compatible con Advanced Microcurrent Technology™*

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