Energel™

WOUND HYDROGEL

Compatible with
V.DOX™ Technology

- EN Instructions for Use
- ES Instrucciones de uso
- FR Mode d'emploi
- PT Instruções de uso

Manufactured for: / Fabricado para: /
Fabriqué pour: / Fabricado para:
Vomaris Wound Care, Inc.
1911 East Fifth Street
Tempe, AZ 85281 USA
(866) 496-8743 (480) 921-4948
www.vomaris.com

© 2020 Vomaris. All rights reserved. Energel, V.Dox, Vomaris, and Vomaris logo are trademarks of Vomaris Innovations, Inc.

IFU-111 Rev. A

Energel™

Wound Hydrogel

Compatible with V.Dox™ Technology

Description

Energel™ Wound Hydrogel is a sterile and water-soluble gel formulated to maintain a moist wound environment and provide moisture to a dry wound. Energel Wound Hydrogel may be used to moisten V.Dox Technology dressings.

Indications

Energel Wound Hydrogel is intended to cover a wound to protect against abrasion, friction, desiccation, and contamination.

Application

1. Cleanse wound and surrounding skin prior to application.
2. Apply enough Energel to cover the wound base.
3. Alternatively, a dressing may be moistened with a thin, even layer of Energel and placed onto the wound.
4. Apply appropriate cover dressing(s) to help manage wound drainage and maintain a moist wound environment.

Removal and Disposal

- Gently remove gel from wound site.
- Discard after single use.

Contraindications

None known.

Storage

- Store in a cool dry place.
- Store away from direct sunlight.

Warnings and Precautions

- For external use only.
- Bleeding should be controlled before application of cover dressings.
- Do not use on patients with known sensitivity to parabens.
- Discard product if packaging is damaged or evidence of contamination is present.
- Single use only. Once opened, this product should not be reused due to risk of contamination.

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