

VOMARIS™ FACE MASK

with V.DOX™ TECHNOLOGY



TECHNOLOGY RESEARCH LED BY THE
INDIANA UNIVERSITY SCHOOL OF MEDICINE

INSTRUCTIONS FOR USE

Intended Use

Vomaris™ Face Mask with V.Dox™ Technology is a source control mask for general public and healthcare personnel use to cover the nose and mouth to help prevent or slow the spread of infection or illness from the SARS-CoV-2 virus during the COVID-19 public health emergency.

Washable and reusable up to 20 times, according to instructions. Failure to follow these instructions may result in altered performance.

Note: Product is not made with natural rubber latex.

HOW TO USE

Application:

Before use, see next page for general guidelines.



Hold mask by ear loops.



Place one loop around each ear and fit mask to face.



Pinch metal nose piece to conform and fit comfortably around bridge of nose.



Adjust ear loops to fit snugly against side of face.

Optional:

Moisten the microcell batteries to wear as a source control mask with electric field activation.

Note: Remoisten as desired to maintain battery activation.



Fold mask in half along center seam.



On dotted side of the mask only, moisten with ~5ml (~1tsp) of saline or tap water.



Moisten the other half with ~5ml (~1tsp) of saline or tap water.



Apply mask, dots facing outwards.

General Guidelines:



Wash hands before putting on and immediately after removing mask. Use hand sanitizer if soap and water are not available.



Once fitted, avoid touching mask while wearing it.



Cover both nose and mouth and secure under your chin.



Be careful not to touch your eyes, nose, or mouth when removing mask.



Discard and replace mask when parts show signs of wear and tear.

Washing and Drying:



Hand wash in cold water (75 to 85°F).



Hang to dry.



Do not bleach.



Do not iron.



Do not dry clean.

Face Contact Materials

Inner layer: 100% polyester with Daikin UNIDYNE TG Series: C6 hydrophobic coating.

Ear Loops: Elastic edging: 91% nylon and 9% spandex. Ear loop adjusters: Polyvinyl chloride.

Vomaris Face Mask has been authorized by FDA under an EUA for use as source control by the general public as well as by HCPs in healthcare settings to help prevent the spread of infection or illness during the COVID-19 pandemic.

This product is authorized only for the duration of the Face Mask EUA.

This product has not been FDA cleared or approved.

It is not a surgical mask nor provides liquid barrier protection.

It is not intended for use in clinical settings where: significant exposure to liquid, bodily, or other hazardous fluids may be expected; clinical settings where the infection risk level through inhalation exposure is high; and in the presence of a high intensity heat source or flammable gas.

It should not be used in place of a surgical mask or filtering facial respirator as it does not provide liquid barrier protection, antimicrobial/antiviral protections, respiratory protection, particulate filtration, or protect in high-risk aerosol generating procedures.

It does not prevent COVID-19 infection or influenza.

Made in the U.S.A. Due to the nature of this product, it cannot be returned. All sales are final.



Vomaris Wound Care, Inc.
1911 East Fifth Street
Tempe, AZ 85281 USA
(866) 496-8743 (480) 921-4948

www.vomaris.com